EN

EXCILOR®

Protection against foot infections Protector Spray 3-in-1

Topical spray for the protection against verrucas / plantar warts, athlete's foot and nail fungus.

- Read this instruction leaflet carefully before using Excilor® 3-in-1 Protector Spray
- To achieve the best results please follow the instructions provided below
- Keep this instruction leaflet in a safe place. You may wish to refer to it again
- Should you require any additional information or in case of side-effects, please contact a medical professional

What is Excilor® 3-in-1 Protector Spray and what is it used for?

- Excilor® 3-in-1 Protector Spray is a medical device designed for the protection against verrucas / plantar warts, athlete's foot and nail fungus.
- Once sprayed onto feet and toes Excilor® 3-in-1 Protector Spray instantly creates a superficial barrier that has been specifically proven to:
 - prevent direct contact of micro-organisms with skin and nails
 - create a superficial environment hostile to microbial proliferation
 - protect the skin for a minimum of 8 hours.
- Duration of protection refers to normal conditions and use. To extend protection or in case of intense or prolonged heat, humidity, perspiration or friction, feel free to repeat application as desired.

Excilor® 3-in-1 Protector Spray is invisible, non-greasy and does not stain clothes. Its specifically tested formulation does not require rinsing and leaves the skin clean, moisturized and freshly scented. Whenever desired, it can be washed off easily with any soap. Excilor® 3-in-1 Protector Spray contains a mix of specific preservatives that will effectively inhibit microbial colonization of its superficial barrier on skin and nails.

When to use Excilor® 3-in-1 Protector Spray

- before participating in barefoot activities such as meditating, dancing, martial arts, yoga and pilates, to cite a few, as well as walking barefoot in communal (wet) areas and walking on carpets, mats or rugs in hotels, etc.
- anytime to minimize cross-contamination from shoes and vice versa
- whenever there is a risk of infection

What should you know before applying Excilor® 3-in-1 Protector Spray?

- Excilor® 3-in-1 Protector Spray:
 - is a specifically tested medical device for the protection against verrucas / plantar warts, athlete's foot and nail fungus
 - is invisible, non-greasy and will not stain clothes
 - dries within seconds and it is safe for repeated application
 - does not require rinsing; however, whenever desired, it can be washed off easily with any soap
 - is paraben-free and leaves the skin clean, moisturized and freshly scented

Instructions for use

- Apply a thin layer of Excilor® 3-in-1 Protector Spray on feet and toes
- Repeat as desired
- After use, leave-on or wash off with a skin soap upon your choice
- Should you require any additional information, please contact a medical professional

Warnings

- Read label before use
- Do not apply to open wounds or injured skin
- For external use only
- For intended use only
- For use in the prevention of other infections or on areas different than feet, a medical professional should be consulted
- Do not use in case of hypersensitivity to any ingredient of the formulation
- In case of any adverse effect or abnormal reaction, please wash off immediately and consult a medical professional
- There is no evidence of interaction with other preparations or drugs, nevertheless, do not use in combination with other topical products
- Keep out of reach of children
- Do not spray onto your eyes. If IN EYES: rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing
- Avoid direct inhalation
- Do not use after expiry date indicated on the bottom of the packaging
- Do not use in case of damaged container
- Extremely flammable aerosol. Keep away from heat, sparks, open flames, hot surfaces.
- No smoking. Pressurized container: do not pierce or burn, even after use; may burst if heated.
- Do not spray on an open flame or other ignition source. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F.

Store at temperatures not exceeding 25°C.

Composition:

Aqua, alcohol, panthenol, glycerol, vitamin B6 HCl, sodium ascorbyl phosphate, xylitol, shikimic acid, perfume, octyl acrylamide/acrylates/butyl amino ethyl methacrylate copolymer, lactic acid, chlorhexidine digluconate, piroctone olamine, dimethyl ether.

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